

HOW TO SEND A CASE VIA SIRONA CONNECT TO RAYVEN DENTAL LAB.

(WHEN SCANNING IN THE CEREC CONNECT SW IT'S THE SAME PROCESS AS IN THE CEREC SW.)

Once you have added the patient's info:

1. Choose weather it is a "Single Restoration" or "Bridge Restoration"
2. Choose the tooth #'s
3. Choose the material shade

Now you are ready to scan!

Scan:

1. Lower Jaw
2. Upper Jaw
3. Buccal
4. Add Category if needed (biocopy, Bioreference ,etc.)
5. Choose your margins

Now you're ready to send to: Rayven Lab

1. Enter your username and password
2. Choose RAYVEN LAB
3. Enter return date and time
4. Enter patient's gender and age
5. Add any additional information or files

Finally you will click "submit" and verify your user name and password!