

HOW TO SUBMIT A CASE TO RAYVEN DENTAL LAB USING MEDIT

1. ENTER YOUR LOG IN INFORMATION ON THE MEDIT LINK LOGIN PAGE
2. OPEN YOUR CASE BOX AND CLICK ON THE BLUE "NEW CASE" BUTTON LOCATED IN THE UPPER RIGHT CORNER OF YOUR SCREEN.
3. FILL OUT CASE INFORMATION AND CLICK THE "REGISTER SCAN" BUTTON.
4. NEXT YOU WILL BE TAKEN TO THE SCANNING SCREEN, ONCE THERE SELECT THE "ARCH" TAB.
5. ONCE YOU ARE ON THE "ARCH" TAB, YOU WILL NEED TO CLICK ON THE UPPER ARCH AND THEN SELECT "ORTHODONIC" AS TYPE OF SCAN. REPEAT THE PROCESS FOR THE LOWER ARCH, AND THEN CLICK THE SCAN BUTTON.

CLEAR CORRECT APP.

1. GO TO THE SIDE MENU FOUND ON THE LEFT SIDE OF THE SCREEN AND CLICK ON "APP BOX"
2. SELECT "CLEAR CORRECT INTERGRATION" AND THEN CLICK INSTALL. ONCE THE APP. HAD BEEN INSTALLED YOU WILL RECIVE A POP UP NOTIFICATION INFORMING YOU THE APP. WAS SUCCESSFULLY INSTALLED.

SENDING SCANS TO RAYVEN DENTAL LAB

1. CLICK ON THE "CASE BOX" ICON ON THE SIDE MENU AND THEN OPEN THE CASE YOU WANT TO SUBMIT.
2. ONCE YOU ARE ON THE CASES PAGE CLI